MioCARE⁺™


A200/A300 Series

Quick Start Guide	en	01
Schnellstartanleitung	de	15
Guía de inicio rápido	es	29
Guide de mise en route	fr	43
Guida rapida	it	57
Snelstartgids	nl	71
Guia Rápido de Iniciação	pt	85
Instrukcja szybkiego uruchomienia	pl	99
Hızlı Başlangıç Kılavuzu	tr	113
クイック スタート ガイド	ja	127
快速使用指南	tc	138

Welcome

Thank you for purchasing a MioCARE product. This Quick Start Guide guides you through the successful setup of your device and familiarises you with the basic skills of using the device.

Getting to know your device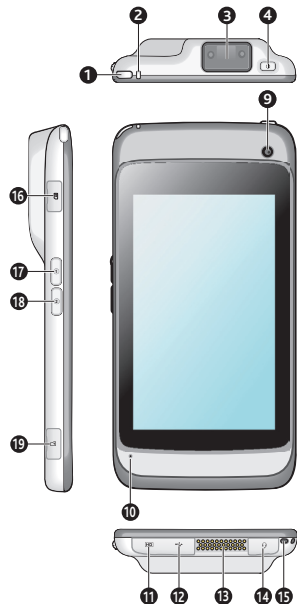


1. LED Indicator 1
Barcode reader status*:
Green/Amber
2. LED Indicator 2
Battery charge status:
Green/Amber
3. Barcode Reader*
4. POWER Button
5. NFC (Near Field Communication) Sensor
6. Speaker
7. Rear Camera
8. Flash
9. Front Camera
10. Microphone
11. HD Connector
12. Mini-USB Port
13. Dock Connector
14. Headphone Socket
15. Lanyard Holder
16. Micro SIM Card Slot*
17. Volume Up Button / Barcode Reader Button*
18. Volume Down Button / Programmable Application Button
19. Memory card slot

* For selected models only.

Getting started

Charging the battery

When charging the battery for the very first time, you should charge it for at least 8 hours (using the AC charger) with your device turned off. To charge the battery, plug the home charger cable into the Mini-USB port on the bottom of your device and the mains power charger into the power socket*.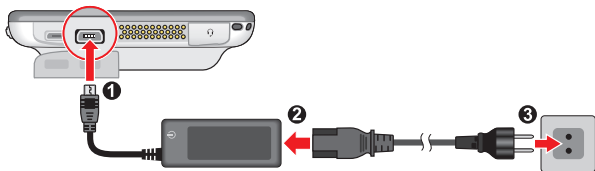

* The plug type varies with the region of your purchase.

The LED indicator will blink **Amber** while the battery power is less than 15%. During charging, the LED indicator on the device lights up to indicate the battery charge status:

Green – battery full;

Amber – battery charging.

CAUTION! For optimal performance of the lithium battery, note the following:

- Do not charge the battery where the temperature is high (e.g. in direct sunlight). The battery will stop charging when the ambient temperature is less than -10°C (14°F) or more than 60°C (140°F).
- Recharge the battery when it is nearly discharged. When recharging, make sure that the battery is fully charged. Doing so can extend the battery life.
- If the device has not been used for over a month or if you find the discharge/recharge time has been shortened, be sure to fully discharge the battery first before recharging it. You are also advised to fully discharge and recharge the battery once every 1–2 months.
- Failure to follow the battery usage instructions can cause damage to your device, battery and even bodily injury or property damage and will void the warranty.

Installing the memory card and SIM card

1. Open the cover on the left side of the device to locate the Micro SIM card slot* (a) and MicroSD card slot (b).
2. Follow the instruction to install the cards (not supplied).
3. Replace the cover and ensure the cover is securely closed.

* For selected models only.


Notes:

- The device supports up to 64GB MicroSD memory cards. However, it does not guarantee the device's compatibility with memory cards from all manufacturers.
- While installing a card, do not apply pressure to the centre of the card. Do not use your finger or any metal objects to touch and scratch the contact part of the card.
- To remove a card, gently push the top edge of the card inwards to release it and then pull it out of the slot.

Connecting your device to a computer

Connect the Mini-USB end of the USB cable to your device (1) and the other end to a USB port in your computer (2). Once connected, you will be able to transfer the files between your computer and device easily through MTP*.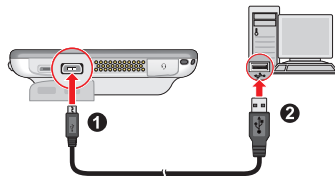


* MTP (Media Transfer Protocol) allows media files (including music, pictures, and other files) to be transferred automatically to and from portable devices. However, for Windows XP users, it is required to install Windows XP Service Pack 3 and Windows Media Player 10 (or higher).

Operating basics

Using the POWER button

- **To turn on the device**

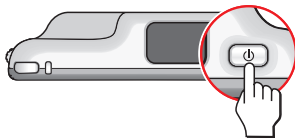Press and hold the POWER button until the device turns on. After the boot up screen, the Home screen will be displayed and the device is in Lock mode by default.

Drag the Lock icon ( > ) to unlock the system. You can now use your device.

- **Sleep mode**

Your device will automatically enter **Sleep mode** after 1 minute (by default) of inactivity. You can also manually switch to Sleep mode by pressing the POWER button briefly.

To resume from Sleep mode, briefly press the POWER button.

- **Aeroplane mode**

When you are in an aeroplane or in the area where the use of wireless radio is prohibited, enable **Aeroplane mode**. Many of your devices applications will continue to work (e.g. playing games or listening music) but all wireless radios, including wireless network and Bluetooth, on the device are turned off – it cannot send or receive data.

To enter Aeroplane mode, press and hold the POWER button and select **Aeroplane mode** from the option menu.

- **To restart / turn off the device**

Press and hold the POWER button and select **Restart / Power off** from the option menu.

- **To shut down the device manually**

Press and hold the POWER button for 5 seconds.

Navigating on the screen


To operate your device, touch the screen with your fingertip. You can perform the following actions:

- **Tap:** Touch the screen once with your fingertip to open items or select onscreen buttons or options.
- **Tap and hold:** Tap and hold on the desired item to open the pop-up menu that contains the available options for the item.
- **Drag:** Tap and hold on the item and then move it to the target position without releasing your fingertip.
- **Slide:** Drag to the left/right (or the top/bottom) to switch to the next/previous item on the list.
- **Multi-touch gestures:** Move your fingertips apart or together on the screen to zoom in or out while viewing a picture or a web page.
- **Rotating the screen:** For most applications, you can automatically change the screen orientation (from portrait to landscape, and vice versa) by turning the device sideways.

Home screen

The Home screen is your starting place for tasks, providing quick access to frequently used applications and settings.

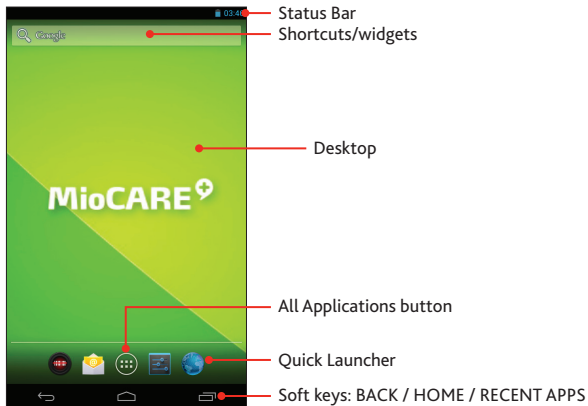Tip: To display the Home screen, tap the HOME button at any time.

Customising your device

You can customise the device through the Settings menu, including personalising the Home screen, changing the display and sound settings, configuring the connection settings, and more.


From the desktop, do one of the following to access the Settings menu:

- Tap the **All Applications** button () > **Settings**.
- Tap  from the Quick Launcher.

Using the Barcode Reader (for selected models only)

By using the Barcode Reader, your device will be able to read the information contained in the barcode.

1. Tap  > **BCR Reader** to launch the Barcode Reader application on your device.
2. Position your device close to the barcode (approximately 5–10 cm), and point the barcode reader sensor to the barcode.
3. Tap the “Scan” button of the application to start scanning; meanwhile, hold your device steadily to keep the barcode within the sensor coverage until the information is displayed on the screen.
4. The LED indicator blinks once to indicate the scanning status:
Green – successful scan.
Amber – scan failed; please try again.
5. To configure the Barcode Reader, tap  > **BCR Settings**.

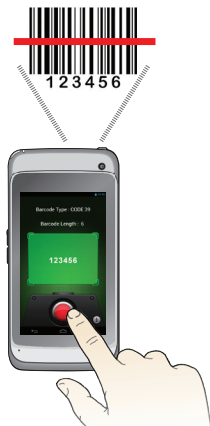Safety precautions

The product covered is an accessory intended for use with Medical Equipment to process data. This device is considered as Medical Device Data System (MDDS), which is designed to serve as a supplementary apparatus for real time or unreal time monitoring on the patient's bio-signals which are generated by other compatible medical monitoring equipment through Wi-Fi, Bluetooth, 3G or USB flash drive. It is hardware product that transfer, store, convert formats, and display medical device data. This device does not modify the data or modify the display of the data, and it does not by itself control the functions or parameters of any other medical device. MDDS are not intended to be used for active patient monitoring.

Caution:

- The equipment provides 1 MOOP (means of operator protection) during the use of car charger that could be touched by the operator but shall not be touched by the patient.
- The device has I/O ports for signal communication. The LED flash light is for assisting the exposure when using camera.

About the charger

- Type of protection against electric shock: Class I equipment.
- Only use the specified power supply: SINPRO MPU16A-102
Input: Universal 100-240 VAC / 47-63 Hz; output: +5 VDC / 2.6A

About charging

- Use only the charger supplied with your device. Use of another type of charger will result in malfunction and/or danger.
- **WARNING!** To avoid the risk of electric shock, this device must only be connected to supply mains with protective earth.
- Do not use the charger in a high moisture environment. Never touch the charger when your hands or feet are wet.
- Allow adequate ventilation around the charger when using it to operate the device or charge the battery. Do not cover the charger with paper or other objects that will reduce cooling. Do not use the charger while it is inside a carrying case.
- Connect the charger to a proper power source. The voltage requirements are found on the product case and/or packaging.
- Do not use the charger if the cord becomes damaged.

- Do not attempt to service the unit. There are no serviceable parts inside. Replace the unit if it is damaged or exposed to excess moisture.
- To fully disengage the power to the unit, please disconnect the power cord from the AC inlet.

About the battery

CAUTION! This unit contains a non-replaceable internal Lithium Ion battery. The battery can burst or explode, releasing hazardous chemicals. To reduce the risk of fire or burns, do not disassemble, crush, puncture, or dispose of in fire or water.

- Use a specified battery in the equipment.
- Important instructions (for service personnel only)
 - Caution: Risk of explosion if battery is replaced by an incorrect type. Dispose of used batteries according to the instructions.
 - Replace only with the same or equivalent type recommended by the manufacturer.
 - The battery must be recycled or disposed of properly.
 - Use the battery only in the specified equipment.

For more information about the device (including the specifications and features, environmental conditions, and more), visit <http://miocare.mio.com/>

Symbols and markings

This appendix provides information about the symbols that are used on the MioCARE series, its accessories or packing.

Class II product: Laser Radiation
Do not stare into beam.

To prevent possible hearing damage, do not listen at high volume levels for long periods.

Follow operating instruction when using device.

Direct current

For more information

Support

For initial support, contact your IT department or your local authorised dealer.

Occasionally, the manufacturer may offer free system updates to improve user experience. Please regularly check the website <http://miocare.mio.com/> for more information.

IP54

The IEC 60529/IP54 is a European system of test specification standards for classifying the degree of protection provided by the enclosures of electrical equipment. An IP54 designation means that the unit withstands dust protected (Ingress of dust is not entirely prevented, but it must not enter in sufficient quantity to interfere with the satisfactory operation of the equipment; complete protection against contact), and water splashing for up to 5 minutes (water volume: 10 litres per minute, Pressure: 80-100kPa). The manufacturer guarantees this grade provided the battery door and all jack covers are properly and securely closed. This device is not IPX7/IPX8 grade; immersion in water or water-pressure such as washing the unit with running water may cause damage to the unit and voids warranty.

Caring for your device

Taking good care of your device will ensure trouble-free operation and reduce the risk of damage to your device.

- Keep your device away from excessive moisture and extreme temperatures.
- Avoid exposing your device to direct sunlight or strong ultraviolet light for extended periods of time.
- Do not place anything on top of your device or drop objects on your device.
- Do not drop your device or subject it to severe shock.
- Do not subject your device to sudden and severe temperature changes. This could cause moisture condensation inside the unit, which could damage your device. In the event of moisture condensation, allow the device to dry out completely before use.
- The screen surface can easily be scratched. Avoid touching it with sharp objects. Non-adhesive generic screen protectors designed specifically for use on portable devices with LCD panels may be used to help protect the screen from minor scratches.
- Never clean your device with it powered on. Use a soft, lint-free cloth to wipe the screen and the exterior of your device.
- Water or rubbing alcohol – A piece of cloth a bit with some water or rubbing alcohol and rub it on the unit.

- Do not use paper towels to clean the screen.
- Never attempt to disassemble, repair or make any modifications to your device. Disassembly, modification or any attempt at repair could cause damage to your device and even bodily injury or property damage and will void any warranty.
- Keep your device and its accessories out of the reach of small children or animals.
- Do not store or carry flammable liquids, gases or explosive materials in the same compartment as your device, its parts or accessories.
- To discourage theft, do not leave the device and accessories in plain view in an unattended vehicle.
- Do not leave the device and its accessories in a vehicle in direct sunlight. Overheating may damage the device.

Troubleshooting

Problem	Solution
Power does not switch on when using battery power.	The remaining battery power may be too low to run your device. Charge the battery.
Screen responds slowly.	Make sure that your device is not running out of battery power. If the problem still persists, reset your device.
Screen freezes.	Reset your device.
Screen is hard to read.	Make sure that the backlight of the display is set to a high-enough brightness.
Cannot establish a connection with a computer.	<ul style="list-style-type: none"> • Make sure that your device and your computer are both switched on before trying to establish a connection. • Make sure that the cable is securely plugged into the USB port on your computer and on your device. Connect the USB cable directly to your computer—do not run the cable through a USB hub. • Reset your device before connecting the cable. Always disconnect your device before you restart your computer.

Note:

- If you encounter a problem you cannot solve, contact an authorised service centre for assistance.
- **WARNING!** Do not modify this device without authorization of the manufacturer.
- Never open the device. For safety reasons, only qualified service personnel should open the device.

Regulatory information (CE)**CE 2200 ⚠**

Note: For regulatory identification purposes, your device is assigned a model number of N435.

This device complies with the R&TTE Directive (1999/5/EC), the EMC Directive (2004/108/EC), and the Low Voltage Directive (2006/95/EC) issued by the Commission of the European Community. - issued by the Commission of the European Community. Compliance with these directives implies conformity to the following European Standards:

EN 301 489-1: Electronic compatibility and Radio spectrum Matters (ERM), Electromagnetic Compatibility (EMC) standard for radio equipment and services; Part 1: Common technical requirements.

EN 301 489-3: Electromagnetic compatibility and Radio spectrum Matters (ERM); ElectroMagnetic Compatibility (EMC) standard for radio equipment and services; Part 3: Specific conditions for Short-Range Devices (SRD) operating on frequencies between 9 kHz and 246 GHz

EN301489-17: Electronic compatibility and Radio spectrum Matters (ERM), Electromagnetic Compatibility (EMC) standard for radio equipment and services; Part 17: Specific conditions for 2.4 GHz wideband transmission systems and 5 GHz high performance RLAN equipment.

EN 300 328: Electromagnetic compatibility and Radio spectrum Matters (ERM); Wideband Transmission systems; Data transmission equipment operating in the 2.4 GHz ISM band and using spread spectrum modulation techniques; Harmonized EN covering essential requirements under article 3.2 of the R&TTE Directive.

EN 301 893: Broadband Radio Access Networks (BRAN); 5 GHz high performance RLAN; Harmonized EN covering the essential requirements of article 3.2 of the R&TTE Directive

EN 300 440: Electromagnetic compatibility and Radio spectrum Matters (ERM); Short range devices; Radio equipment to be used in the 1 GHz to 40 GHz frequency range; Part 2: Harmonized EN covering the essential requirements of article 3.2 of the R&TTE Directive

EN 302 291-1: Electromagnetic compatibility and Radio spectrum Matters (ERM); Short Range Devices (SRD); Close Range Inductive Data Communication equipment operating at 13,56 MHz; Part 1: Technical characteristics and test methods

EN 302 291-2: Electromagnetic compatibility and Radio spectrum Matters (ERM); Short Range Devices (SRD); Close Range Inductive Data Communication equipment operating at 13,56 MHz; Part 2: Harmonised EN under article 3.2 of the R&TTE Directive

EN 50566: Product standard to demonstrate compliance of radio frequency fields from handheld and body-mounted wireless communication devices used by the general public (30 MHz — 6 GHz)

EN 62209-2: Human exposure to radio frequency fields from handheld and bodymounted wireless communication devices — Human models, instrumentation, and procedures

EN 62479: Assessment of the compliance of low power electronic and electrical equipment with the basic restrictions related to human exposure to electromagnetic fields (10 MHz to 300 GHz)

EN 50332: Prolonged listening of music in maximum volume can damage the ears.

This device has been tested to comply with the Sound Pressure Level requirement laid down in the applicable EN 50332-2 standards. Permanent hearing loss may occur if earphones or headphones are used at high volume for prolonged periods of time.

EN 55022: Radio disturbance characteristics

EN 55024: Immunity characteristics

EN 60601: Medical electrical equipment —Part 1-2: General requirements for basic safety and essential performance — Collateral standard: Electromagnetic compatibility — Requirements and tests

EN 61000-3-2: Limits for harmonic current emissions

EN 61000-3-3: Limitation of voltage fluctuation and flicker in low-voltage supply system

IEC 60601-1:2005: Product Safety

For MioCARE A300 series:

EN 301 489-7: Electromagnetic compatibility and Radio spectrum Matters (ERM); ElectroMagnetic Compatibility (EMC) standard for radio equipment and services; Part 7: Specific conditions for mobile and portable radio and ancillary equipment of digital cellular radio telecommunications systems (GSM and DCS)

EN 301 489-24: Electromagnetic compatibility and Radio spectrum Matters (ERM); ElectroMagnetic Compatibility (EMC) standard for radio equipment and services; Part 24: Specific conditions for IMT-2000 CDMA Direct Spread (UTRA and E-UTRA) for Mobile and portable (UE) radio and ancillary equipment

EN 301 908-1: IMT cellular networks; Harmonized EN covering the essential requirements of article 3.2 of the R&TTE Directive; Part 1: Introduction and common requirements

EN 301 908-2: IMT cellular networks; Harmonized EN covering the essential requirements of article 3.2 of the R&TTE Directive; Part 2: CDMA Direct Spread (UTRA FDD) User Equipment (UE)

EN 301 511: Global System for Mobile communications (GSM); Harmonized EN for mobile stations in the GSM 900 and GSM 1800 bands covering essential requirements under article 3.2 of the R&TTE directive (1999/5/EC)

The manufacturer cannot be held responsible for modifications made by the User and the consequences thereof, which may alter the conformity of the product with the CE Marking.

Declaration of Conformity

Hereby, MiTAC declares that this N435 is in compliance with the essential requirements and other relevant provisions of Directive 1999/5/EC.

SAR information

The radio wave exposure guidelines employ a unit of measurement known as the Specific Absorption Rate (SAR), which adopts the limit of 2 W/kg averaged over ten (10) gram of tissue. Your device has been designed to comply with applicable safety requirements for exposure to radio waves.

SAR \leq 0.96 W/kg

Bluetooth

Bluetooth DID D021801

WEEE

This product must not be disposed of as normal household waste, in accordance with the EU directive for waste electrical and electronic equipment (WEEE - 2002/96/EC). Instead, it should be disposed of by returning it to the point of sale, or to a municipal recycling collection point.

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Revision: R01
(12/2014)

Disclaimer

Specifications and documents are subject to change without notice. The manufacturer does not warrant this document is error-free. The manufacturer assumes no liability for damage incurred directly or indirectly from errors, omissions, or discrepancies between the device and the documents.

Notes

Not all models are available in all regions.

Depending upon the specific model purchased, the color and look of your device and accessories may not exactly match the graphics shown in this document.

The screenshots and other presentations shown in this document are for reference only. They may differ from the actual screens and presentations generated by the actual product.

5 6 1 5 N 4 3 5 0 0 1 R 0 1